Carospeed Classic

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Handhavande

Thank you for choosing a

Carospeed Classic from BraunAbility!

The following manual is an important part of the product, providing you with information on how to achieve maximum performance and safe operation. Keep the manual in a safe place so that you can refer to it when necessary.

If you have any questions about your adaptation, please contact your installation engineer. If you want to know more about us and our range of car adaptation solutions visit www.braunability.eu.

Once again, thank you for placing your confidence in our products!

Safety

14 Safety measures

Operation

15 Hill-holder

16 Start and drive off

16 Park the car

17 Option, primary electrical functions

“Product certificate” at the end of the manual.

Safety

Safety measures

Carospeed Classic may only be used by individuals who has a prescription to do so.

Do not use Carospeed Classic as a handle to get inside the car.

Contact an authorized technician if the Carospeed Classic appears to be wiggly sideways.

Carospeed Classic shall be inspected by an authorized technician if a car crash has occurred.

Operation

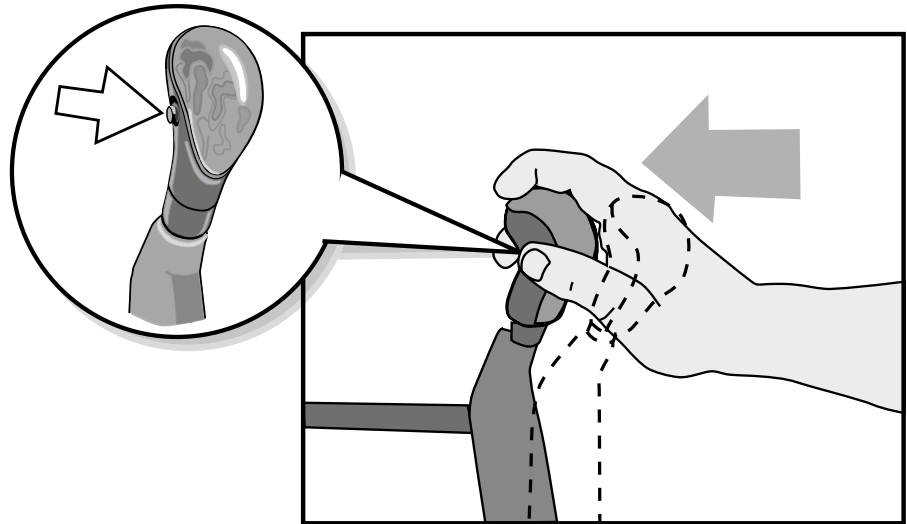
Hill-holder

The hill-holder function enables the user to brake the car at the same time as the gear shift is set. Activate the hill-holder by pressing the button on the left hand side of the handle. It can only be activated while braking the car, when sufficient brake power is achieved and ignition is on.


The hill-holder lets go automatically as soon as the handle is pushed forward (braking).


Never leave the car while the hill-holder is activated.

Start and drive off

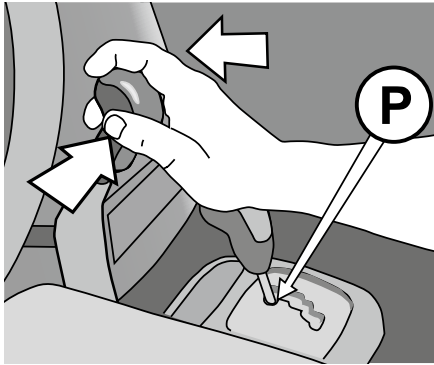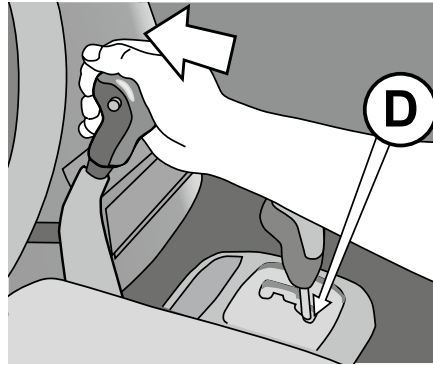
- 1 Start the engine with the gear shift in parking position. Activate the hill-holder. Now the gear shift D or R can be set without the car moving.

- 2 Release the parking brake. Un-activate the hill-holder (press the handle forward and the hill-holder lets go automatically). Now you can start driving.

To brake push the handle forward.
To accelerate pull the handle backwards.

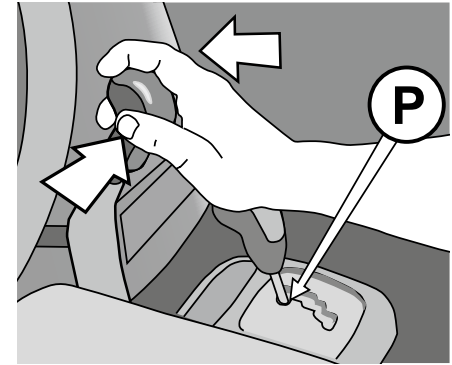Park the car


- 1 Brake the car and activate the hill-holder as soon as the car has stopped. Set the gear shift to parking position (P) and pull the parkingbrake. Un-activate the hill-holder by pushing the handle forward.

The hill-holder shall never be used as substitute for the parking brake.


Option, primary electrical functions

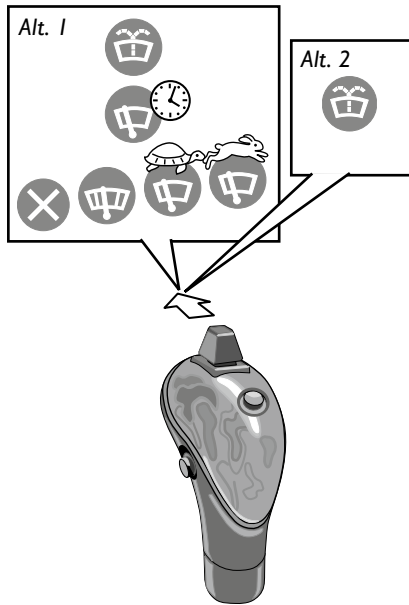
When Carospeed E or TE is being installed the wiper function can be programmed in two different ways.

Quick wipe Press the joystick forward and hold it there until the wipers start to move. Release. After a couple of sweeps they are automatically shut off.

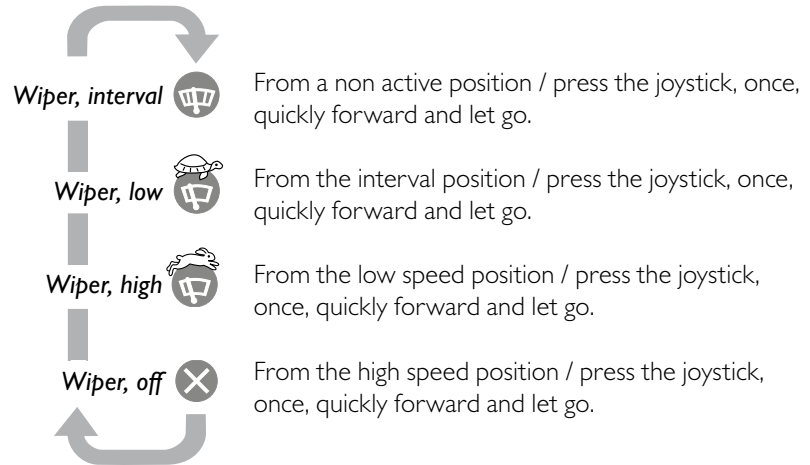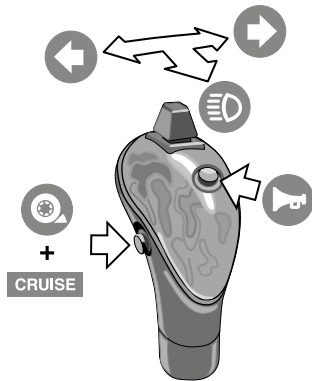Wash and wipe Same as above, but hold the joystick forward until also the washer has started. Keep the joystick there as long as washing is needed. After activation a couple of sweeps will follow.

Flashers, left/right

Press the joystick towards the left/right and let go. The flashing will stop automatically after a short while, unless the brake light is activated, in which case the flashing will continue for a short while after the brake has been un-activated. Alternatively the flashing can be ended manually, by pressing the joystick once again towards the activated side. If one flasher direction has been activated the direction can easily be changed by just pushing the joystick towards the other side, not needing to first shut it off.

Headlight

Press the joystick towards the rear. Dim the headlight by pressing the joystick once again the same way.

Horn

The horn is activated by the separate button on top of the handle.

Hill-holder

If the brake is activated while the Cruise/Hill-holder button is pressed down the Hill-holder is activated.

Cruise

If the brake is not activated and the same button is pressed down the cruise is activated (provided that the vehicle cruise function is activated).

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Phone: +46 302 254 00
E-mail: info@braunability.eu
www.braunability.eu

Declares that the product /

Carospeed Classic, model STD /
T-handle / Electric / Indicator

Conforms to following directives, standards and regulations /

2006/42/EEC
2007/46 EEC
2007/47/EEC
LVFS 2013:11
ECE R-10
ECE R- 21
ISO 3795
ISO/TC22 N 2072
AS 3954.1-1991
AS 3954.2-1991

BraunAbility Europe AB, Stenkullen
2019-04-02

Jan M Jensen
Managing Director

Accredited by SWEDAC as testing laboratory in accordance with ISO/IEC 17025:2005
Quality system certified in accordance with ISO 9001:2015

Illustrations, descriptions and specifications in the user manual are based on current product information.
BraunAbility Europe AB reserves the right to make alterations without previous notice.
© 2019 BraunAbility Europe AB

BraunAbility Europe AB
Åkerivägen 7
443 61 Stenkullen
Sweden

Phone: +46 302 254 00
E-mail: info@braunability.eu
www.braunability.eu