Carospeed Menox

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
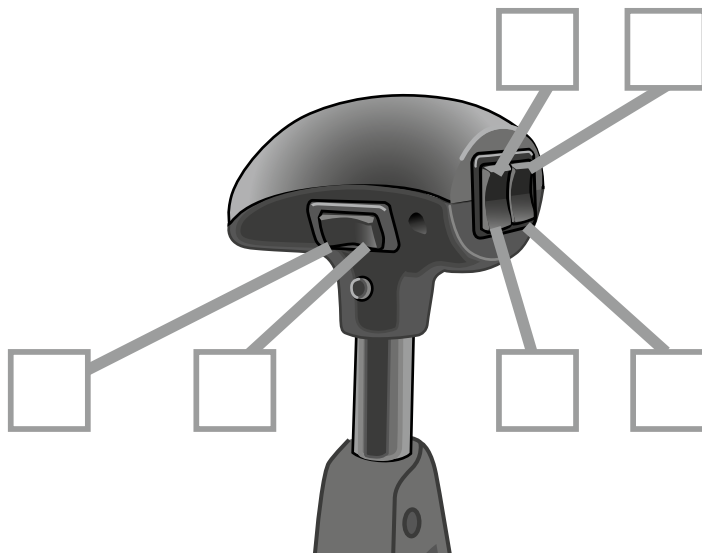
Thank you for choosing a

Carospeed Menox from BraunAbility!

The following manual is an important part of the product, providing you with information on how to achieve maximum performance and safe operation. Keep the manual in a safe place so that you can refer to it when necessary.

If you have any questions about your adaptation, please contact your installation engineer. If you want to know more about us and our range of car adaptation solutions visit www.braunability.eu.

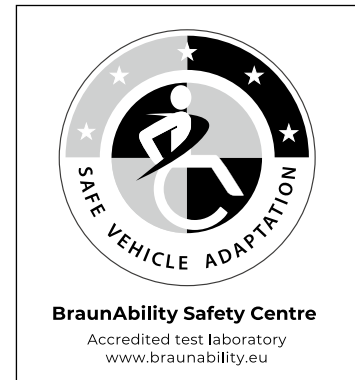Once again, thank you for placing your confidence in our products!

Safety

14 Safety measures

Operation

15 Hill-holder

16 Start and drive off

16 Park the car

17 Cruise control

18 Option, primary electrical functions

“Product certificate” at the end of the manual.

Safety

Safety measures

Carospeed Menox may only be used by individuals who has a prescription to do so.

Do not use Carospeed Menox as a handle to get inside the car.

Contact an authorized technician if the Carospeed Menox appears to be wiggly sideways.

Carospeed Menox shall be inspected by an authorized technician if a car crash has occurred.

Operation

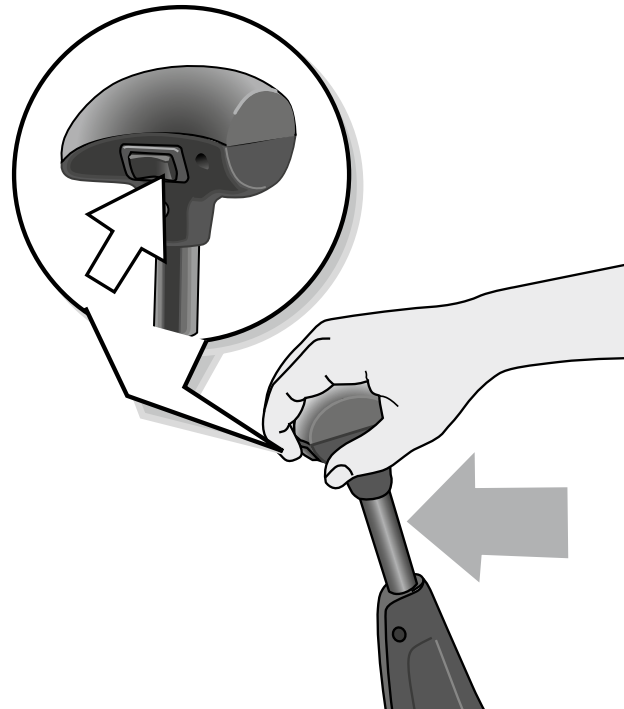
Hill-holder

The hill-holder function enables the user to brake the car at the same time as the gear shift is set. Activate the hill-holder by pressing the button on the back of the handle. It can only be activated while braking the car, when sufficient brake power is achieved and ignition is on.


The hill-holder lets go automatically as soon as the handle is pushed forward (braking).


Never leave the car while the hill-holder is activated.

Start and drive off

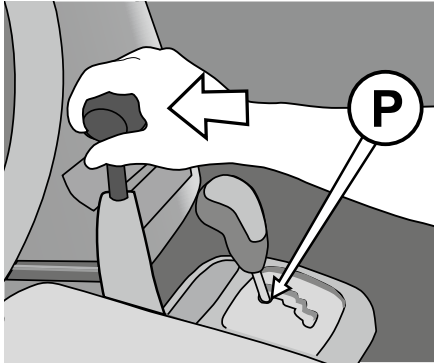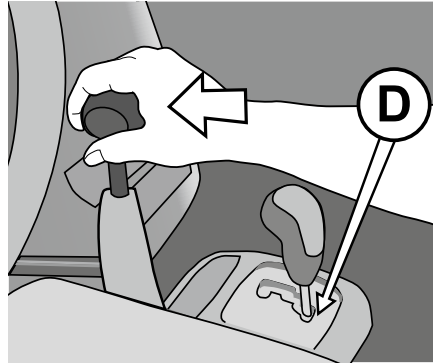
- 1 Start the engine with the gear shift in parking position. Activate the hill-holder. Now the gear shift D or R can be set without the car moving.

- 2 Release the parking brake. Un-activate the hill-holder (press the handle forward and the hill-holder lets go automatically). Now you can start driving.

To brake push the handle forward.
To accelerate pull the handle backwards.

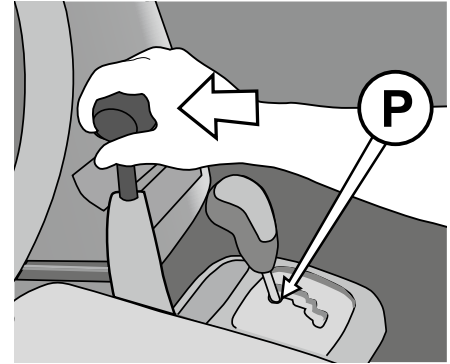Park the car

- 1 Brake the car and activate the hill-holder as soon as the car has stopped. Set the gear shift to parking position (P) and pull the parkingbrake. Un-activate the hill-holder by pushing the handle forward.

The hill-holder shall never be used as substitute for the parking brake.

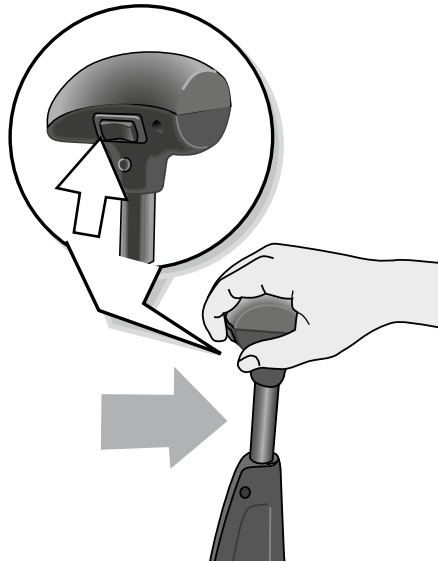
Cruise control

Do not use cruise control in rain or in otherwise slippery conditions.

Do not leave the mechanical cruise control activated when turning off the engine.

Activating the vehicle's own cruise control

The Cruise control is activated by the same switch as the hill-holder, but can only be activated while accelerating. Accelerate to the desired speed. Press the cruise control button once. Now you can let go of the handle and the speed remains the same.

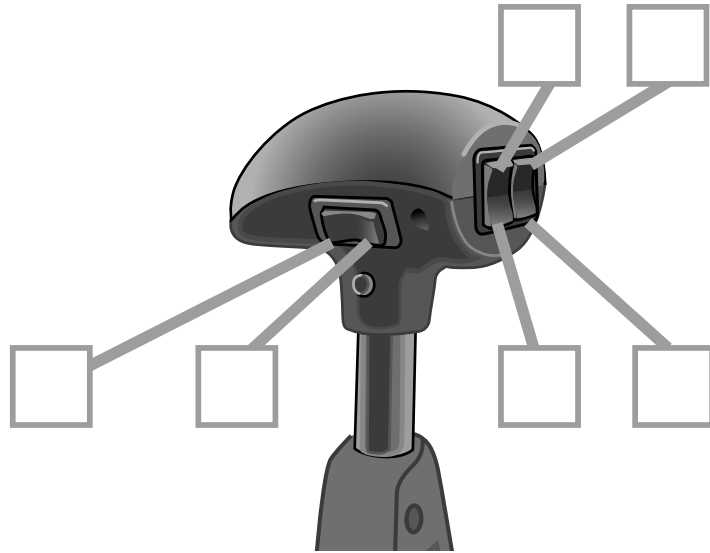
Activating the optional Mechanical Cruise control, for vehicles that lack cruise control function

The mechanical Cruise control is activated by the same switch as the hill-holder, but can only be activated while accelerating. The mechanical cruise control is activated as described in "Activating the vehicle's own cruise control" above. To release just move the handle in any direction.

Option, primary electrical functions

Connections are made according to individual customer needs. The installer is responsible for marking the connections.

1. Wiper 
2. Wash and wipe 
3. Flashers, left/right 
4. Headlight 
5. Horn 
6. Hill-holder 
7. Cruise 
8. Tiptronic +
9. Tiptronic -

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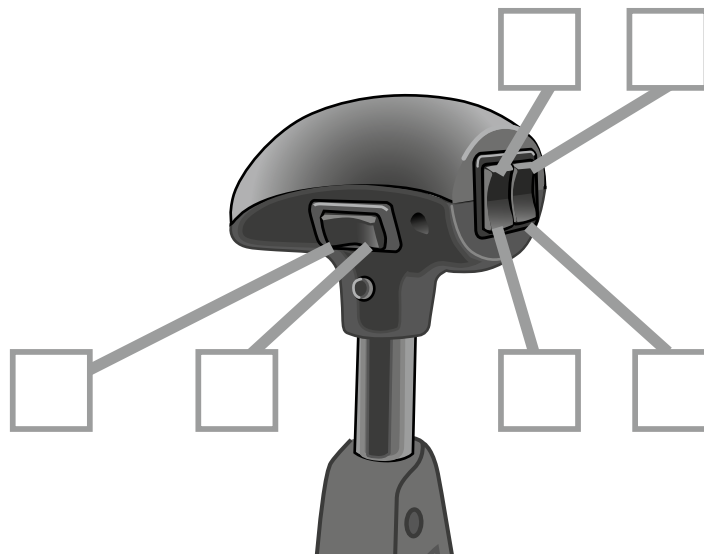
Declares that the product /

Carospeed Menox,
model Left / Right hand

Conforms to following directives, standards and regulations /

2006/42/EEC
2007/46 EEC
ECE R-10
ECE R- 21
ISO 3795
ISO/TC22 N 2072
AS 3954.1-1991
AS 3954.2-1991

BraunAbility Europe AB, Stenkullen
2019-04-02

Jan M Jensen
Managing Director

Accredited by SWEDAC as testing laboratory in accordance with ISO/IEC 17025:2005
Quality system certified in accordance with ISO 9001:2015

Illustrations, descriptions and specifications in the user manual are based on current product information.
BraunAbility Europe AB reserves the right to make alterations without previous notice.
© 2019 BraunAbility Europe AB

BraunAbility Europe AB
Åkerivägen 7
443 61 Stenkullen
Sweden

Phone: +46 302 254 00
E-mail: info@braunability.eu
www.braunability.eu