# Wheelchair lifts

Dual parallel arms lifts

Century XT | Global | Millennium | Vista | Vista Split

SV	4	Bruksanvisning
EN	20	User manual
DE	36	Gebrauchsanweisung
FR	52	Instructions d'utilisation
ES	68	Instrucciones de uso
IT	84	Istruzioni per l'uso

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---

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Thank you for choosing a

# Wheelchair lift from BraunAbility!

The following manual is an important part of the product, providing you with information on how to achieve maximum performance and safe operation. Keep the manual in a safe place so that you can refer to it when necessary.

If you have any questions about your adaptation, please contact your installation engineer. If you want to know more about us and our range of car adaptation solutions visit [www.braunability.eu](http://www.braunability.eu).

Once again, thank you for placing your confidence in our products!

**Safety**

---

- 22 Safety measures
- 23 Lift latches
- 23 Emergency stop

**Operation**

---

- 24 Guidelines for safe lifting
- 25 The lift's controls
- 26 Unfold and lower
- 28 Raise and fold

**Emergency operation**

---

- 30 Unfold and lower
- 31 Raise and fold

**Care instructions**

---

32

**Troubleshooting**

---

33

**Introduction for users**

---

35

“Product certificate” at the end of the manual.

# Safety

## Safety measures

*A Braun Lift is designed and manufactured for use by wheelchair users, or users of other devices, according to the limits indicated in this manual. The platform is only intended for lifting one passenger at a time.*

*Allow only persons familiar with the information contained in this manual to use a Braun Lift.*

*Park the vehicle as level as possible. Ensure that sufficient room is available to operate the Braun Lift concerned. Actuate the vehicle's handbrake before using the lift.*

*Ensure that users of wheelchairs or other devices are positioned inside the platform marked in yellow on the Braun Lift concerned.*

*Avoid any risk of pinching injuries. Ensure that the user is sitting securely fastened on the platform before the lift is operated. Brakes on wheelchairs or other conveyances should be actuated. Keep at a safe distance when the platform is in motion.*

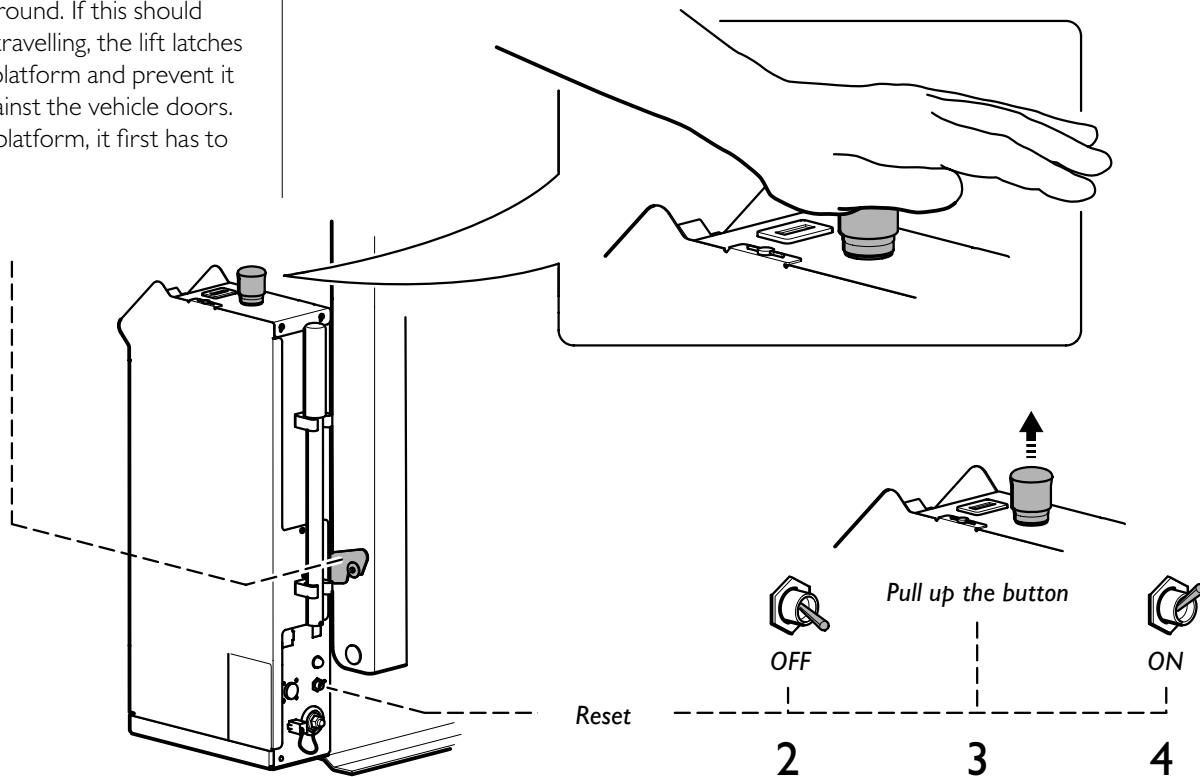
*Any modifications, adjustments or repairs to a Braun Lift may only be performed by authorised technicians*

## Lift latches

In the event of a minor leak in the hydraulic system, the platform will slowly lower to the ground. If this should happen while travelling, the lift latches will catch the platform and prevent it from falling against the vehicle doors. To unfold the platform, it first has to be folded fully.

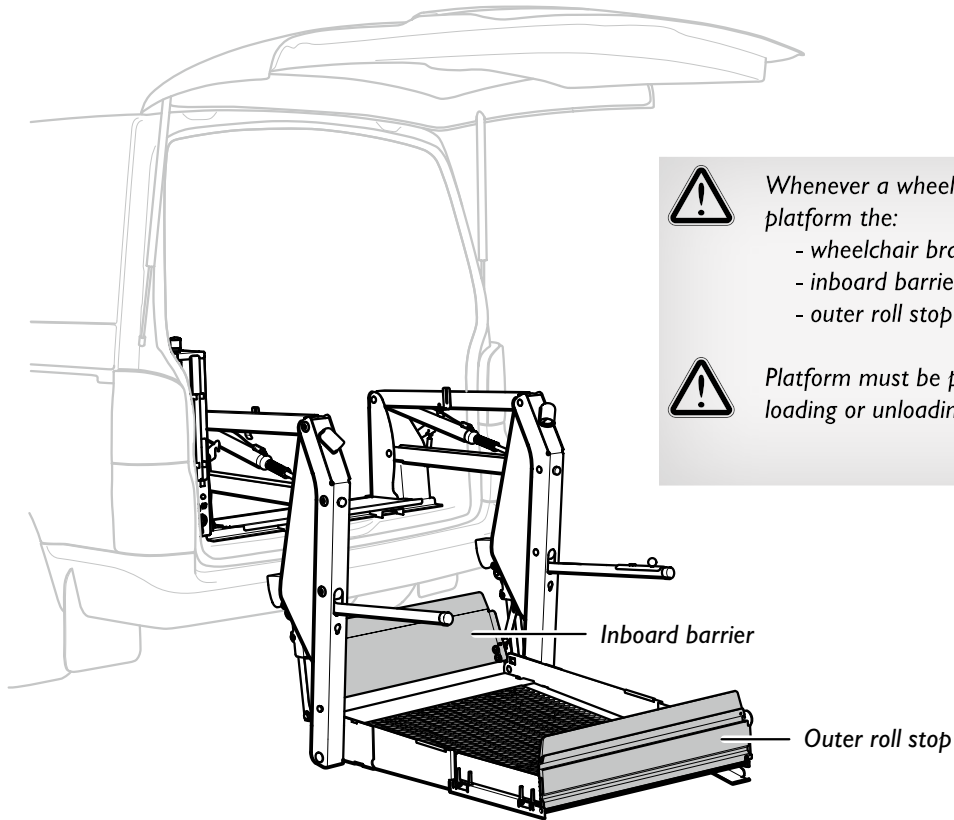
## Emergency stop

Push in the red button on the top of the pump module to stop the lift movement immediately.

## Operation

### Guidelines for safe lifting

*Whenever a wheelchair passenger (or standee) is on the platform the:*

- wheelchair brakes must be locked.*
- inboard barrier and outer roll stop must be up.*
- outer roll stop must be latched.*

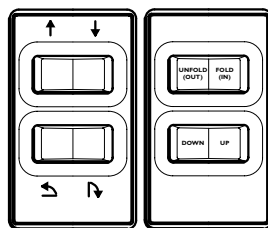
*Platform must be positioned at vehicle floor level when loading or unloading in and out of vehicle.*

## The lift's controls

Choose one of the following three options for operating the lift, see the explanation on the following pages.

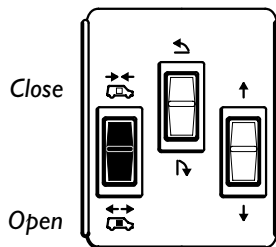
Alt. A Handheld control

Model 1

Model 2  
(Global)

Alt. B Pump module

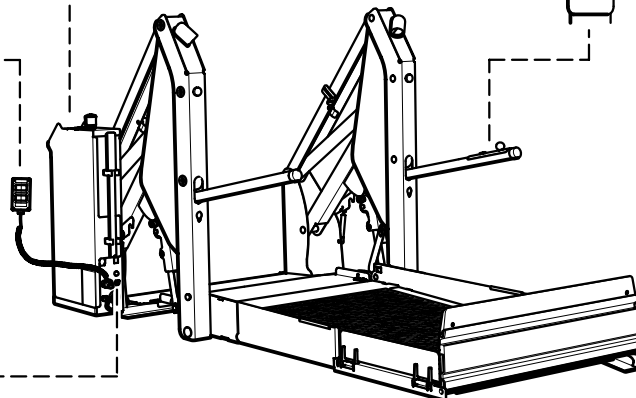
Close and open the doors on the vehicle, manually or using the yellow controller (Close/Open) on the pump module.

Alt. C Handle

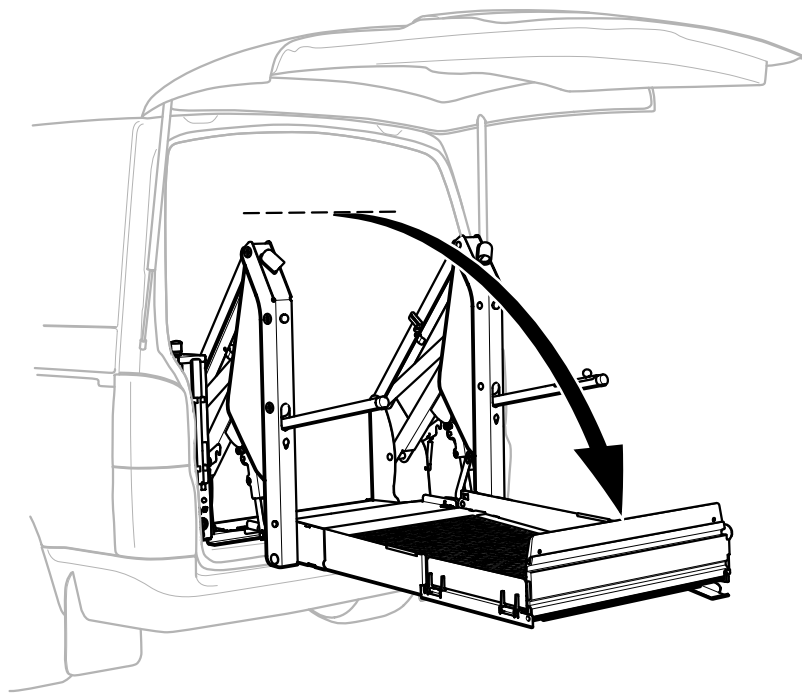
-  Green light
-  Circuit breaker

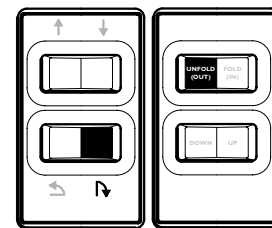
The green light indicates that power to the lift is on. To operate the lift, the circuit breaker must be switched on.

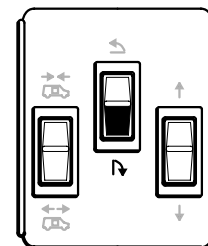
## Unfold and lower

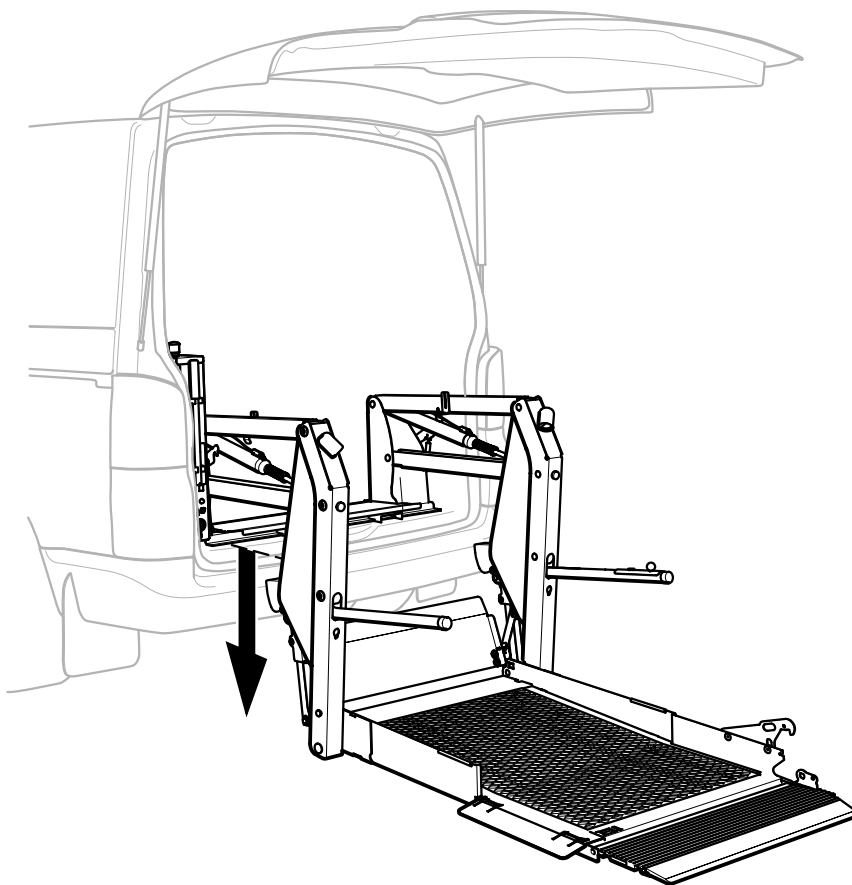
Alt. A Handheld control

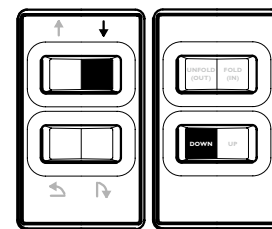
Alt. B Pump module

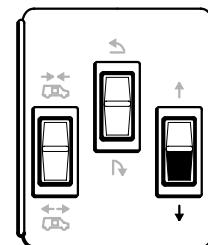
**NOTE** / If the lift does not unfold, the lift latches are actuated, see page 23.  
To unfold the platform, it first has to be folded fully.

Alt. A Handheld control

Alt. B Pump module

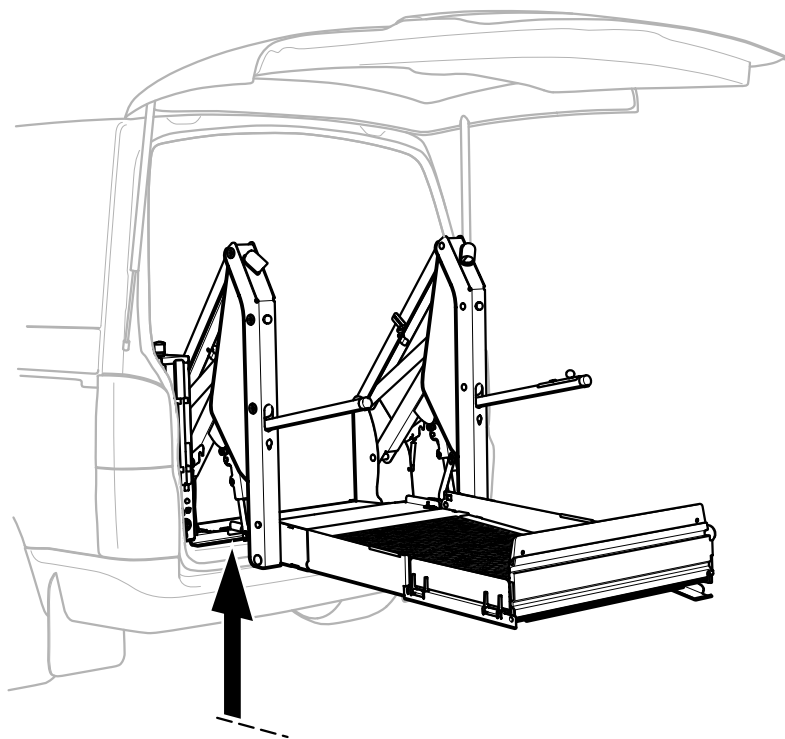
Alt. C Handle

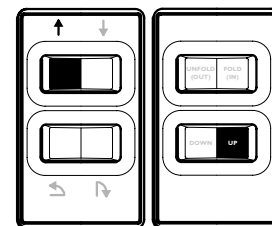
Push the knob back.

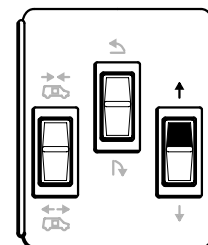
## Raise and fold

Alt. A *Handheld control*

Alt. B *Pump module*

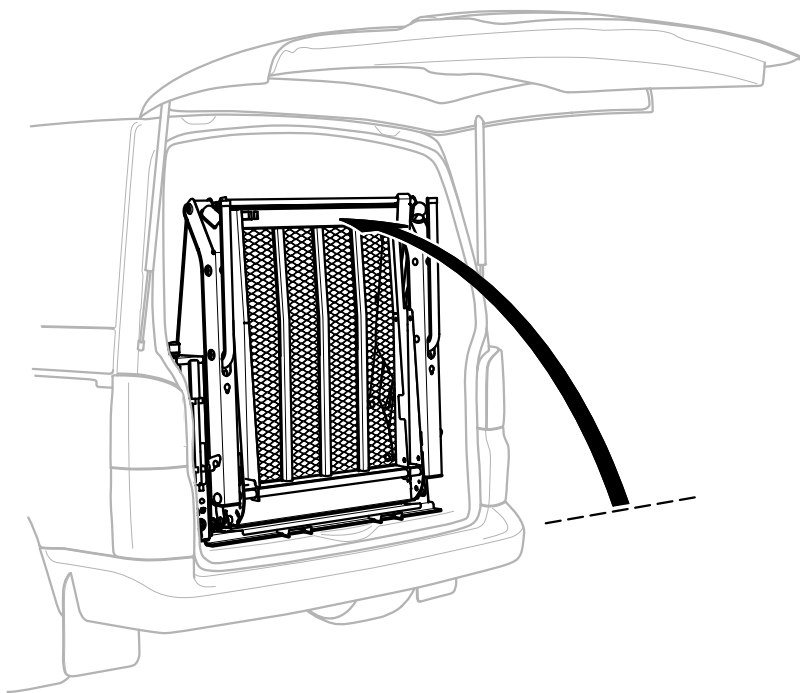
Alt. C *Handle*

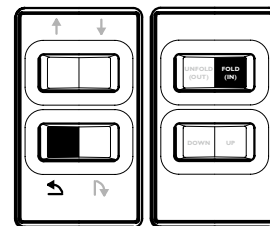
Push the knob forward.

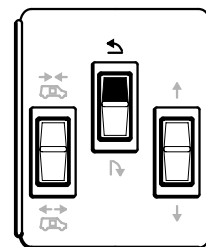
Alt. A *Handheld control*

Alt. B *Pump module*

EN

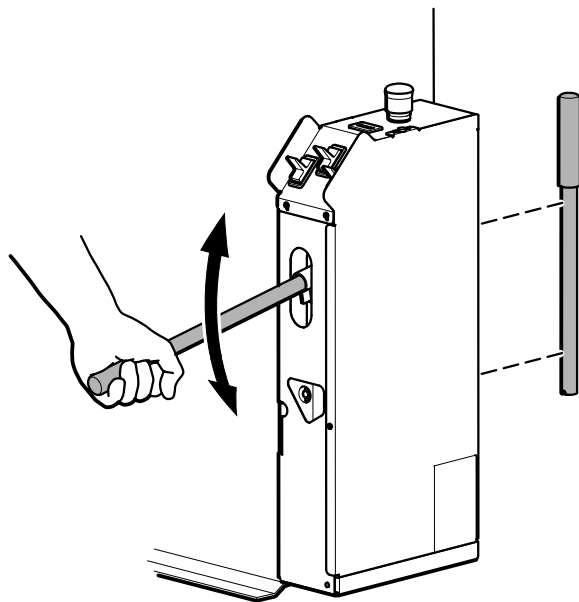
Operation

## Emergency operation

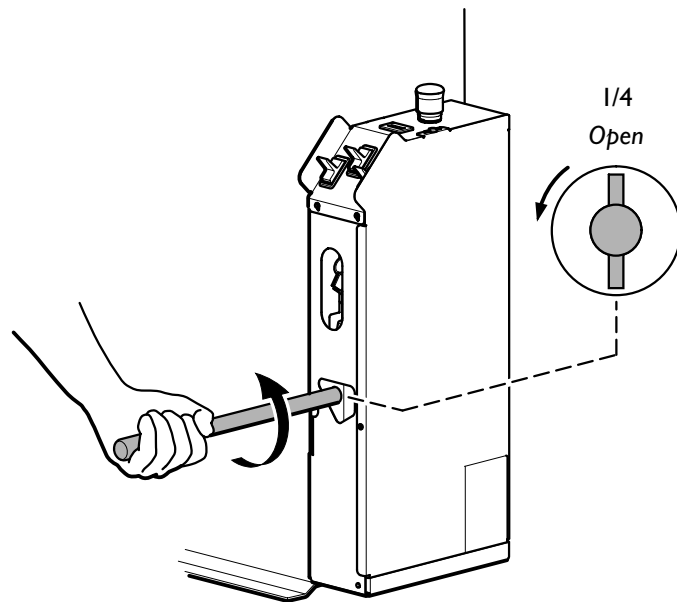
### Unfold and lower

- 1 Only do this element if the lift latches need to be deactivated, see page 23.

Take hold of the handle located on the front of the pump module. Pump until the platform is folded fully.

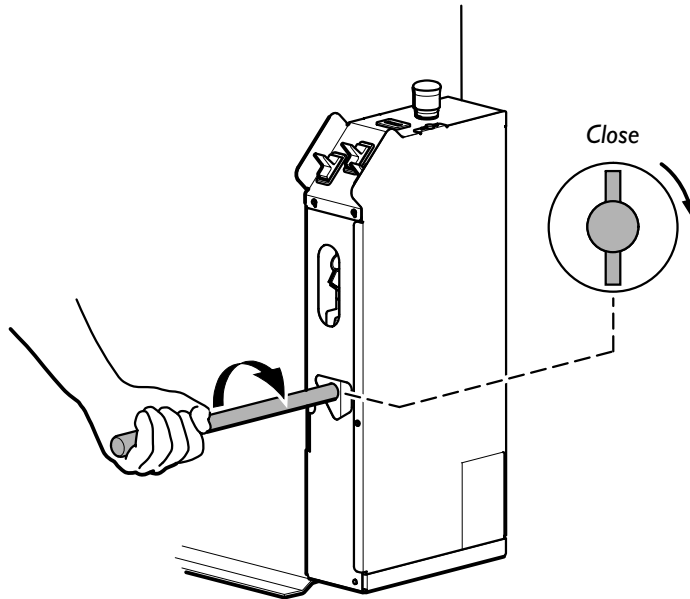
- 2 Turn to open the valve and allow the platform to lower to the ground.

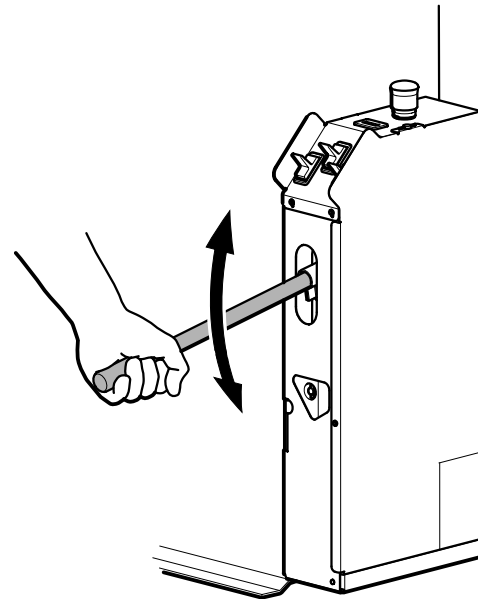
## Raise and fold

1 Turn to close the valve but avoid overtightening.

2 Pump until the platform is folded fully.

## Care instructions

When necessary .....	Clean the lift with a moist cloth and wipe dry.
When the lift is being used .....	Ensure that / <ul style="list-style-type: none"><li>- the lift is moving correctly.</li><li>- inboard and outboard roll stops are working correctly.</li><li>- the commands on the handheld control are working correctly.</li><li>- there are no oil leaks or discordant sounds.</li></ul>
Once a month .....	Ensure that warning labels are in place and are legible.
Once every six months .....	Contact your installer to book a service, see the end of the manual.

- The product documentation must always be available during inspection of the vehicle.
- Always contact your dealer for disassembly.

## Troubleshooting

The platform cannot be lowered.....

Check that /

- the main circuit breaker on the pump module is switched on and that the green light is on.
- the lift is not resting on the lift latches. If these are actuated, the platform first has to be folded fully before it can be unfold, see page 23.
- the fuse on the pump module is working
- the vehicle's battery is fully charged.
- the red emergency stop button is reset.

Operate the lift manually, see page 30. Then contact your installer for an overhaul.

The platform cannot be raised.....

Check that /

- the lift's maximum weight has not been exceeded.
- the oil reaches up to the maximum level marked on the pump module when the lift has been lowered fully.

Operate the lift manually, see page 31. Then contact your installer for an overhaul.

Inboard and outboard roll stops.....  
are not working correctly.

Contact your installer for an overhaul.

The lift is leaking oil or causing.....  
discordant sounds while it is moving.

Contact your installer for an overhaul.

Please contact your installation engineer if you require further assistance or need to ask questions about warranty. We recommend keeping the following information to hand at all times /

Contact details for the installation engineer /

Model /

Serial number /

Year of manufacture /

Date of purchase /

Notes /

---

---

---

---

---

---

---

---

## Introduction for users

BraunAbility Europe AB would like you, as a user, to feel secure and comfortable with your new product. For that reason, it is your installer's duty to go through the following with you /

Safety  
Operation  
Emergency-mode operation

If any questions about the product should come up later on, you are of course very welcome to contact your installation engineer again.

The above chapters have been gone through /

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# Product certificate

## Manufacturer /

The Braun Corporation  
631 West 11 th Street, Winamac  
IN 46996 United States

## Authorized representative and importer into EU /

BraunAbility Europe AB  
Åkerivägen 7  
443 61 Stenkullen  
Sweden

Phone: +46 302 254 00  
E-mail: [info@braunability.eu](mailto:info@braunability.eu)  
[www.braunability.eu](http://www.braunability.eu)

## Declares that the product /

Braun Dual Parallel Arms lift,  
Model: All versions 12/24 volts system

## Conforms to following directives, standards and regulations /

2006/42  
ECE R-10, ECE R-17  
EN 1756-2:2004 + A1 2009  
EN 50489: 2010  
ISO EN 12100-1/2  
ISO 7637-2: 2011  
CISPR 25:2008  
ECE R-107 Rev. 5  
EN 60204  
ISO 14121-2: 2012  
EN 11202  
REACH, GADSL, RoHS

BraunAbility Europe AB, Stenkullen  
2019-04-02

Jan M Jensen  
Managing Director

Accredited by SWEDAC as testing laboratory in accordance with ISO/IEC 17025:2005  
Quality system certified in accordance with ISO 9001:2015

Illustrations, descriptions and specifications in the user manual are based on current product information.  
BraunAbility Europe AB reserves the right to make alterations without previous notice.  
© 2019 BraunAbility Europe AB

**BraunAbility Europe AB**  
Åkerivägen 7  
443 61 Stenkullen  
Sweden

Phone: +46 302 254 00  
E-mail: [info@braunability.eu](mailto:info@braunability.eu)  
[www.braunability.eu](http://www.braunability.eu)